

REF 33.100.500 DENT-NOI xxxxxx Expiry date 20xx-xx

Rev. 14 (2023-05) MAPA GmbH Industriestraße 21-25, 27404 Zevern, Germany

www.billy-boy.de



1 condom

# BILLY BOY®

1 condom BILLY BOY Extra Lubricated made from natural rubber latex with lubricant and reservoir. Plain, cylindrical shape, transparent. Width of the plain condom: 52 mm.  
BILLY BOY condoms are medical products for contraception and preventing infections. Internationally recognised production standards and constant laboratory testing form the basis of the good quality and safety of BILLY BOY condoms. Condoms are dermatologically tested and each condom is as well electronically tested.  
However, no contraceptives completely eliminate the risks of pregnancy or infection. Do not use condoms with oil-based or greasy lubricant.

Please note the following information symbols on the packaging:



Please read the instructions for use carefully.



Protect condoms from direct sunlight and heat.



Store condoms in a dry place.



Do not use the condom if the wrapper is damaged.



Each condom should only be used once.



Condoms made from natural rubber latex.



Medical Device (EU) 2017/745



Manufacturer



Lot number



Expiry date

### Instructions for use



#### 1. Opening

Carefully tear open the condom wrapper at the marking ("open here"). The wrapper can thus be torn open cleanly without damaging the condom. Caution: The condom may be damaged by sharp objects, jewellery or fingernails. Do not use the condom if the wrapper is damaged or already open.



#### 2. Before intercourse

Hold the condom teat between the thumb and forefinger, making sure that no air is trapped inside. Pull back the foreskin and unroll the condom all the way down the erect penis before any contact of the penis with the body of your partner. If the condom does not unroll easily, it may be on backwards. In this case, do not turn it around for use as your partner may come into contact with body secretions. Throw it away and use a new condom, making sure that it is on the right way for unrolling. If you notice that the condom slips or tightens, check it for potential damage as this may cause a tear.



#### 3. After Intercourse

Withdraw the penis before it becomes flaccid! Make sure to hold the condom against the base of the penis to prevent the condom from slipping off.

### Important notes:

Condoms should always be handled with care. Please check the expiry date on the package and the condom-wrappers - do not use the condom if expired! Store condoms in a cool and dry place and protect it from direct sunlight.

Condoms do not interfere with the human organism and have no side effects. There is, however, a possibility of intolerances occurring, which could involve allergic reactions (e.g. latex allergy) or even anaphylaxis.

Should you encounter any discomfort or skin irritation, stop using the condom and consult your doctor. If used correctly (see instructions for use), BILLY BOY condoms provide protection from pregnancy and transmission of diseases. However, no contraceptives completely eliminate the risks of pregnancy or infection. If the condom has torn or broken during use, seek medical advice as soon as possible, but within 72 hours.

Please report any serious incident in relation to the device to the manufacturer and your national competent authority. Used condoms should be disposed of in a waste bin, not down the toilet. Wash penis and hands before resuming intimate contact. Condoms are designed to be used only once.

### Attention:

Do not use condoms with oil-based or greasy lubricant (e.g. vaseline, baby-oil, bodylotion, massage-oils, cooking-oils). Those affect the condom and cause it to tear quickly. If using a lubricant, always ensure that the label states that it is suitable for use with condoms. Prior to using lubricants, please read the instructions for use on the packaging. Never apply lubricant directly onto the penis or the inside of the condom as this will increase the risk of the condom slipping off or tearing. When using condoms for anal intercourse, you should always also apply lubricant (e.g. BILLY BOY lubricant gel) to the condom as this

reduces the risk of the condom slipping off or tearing. If vaginal suppositories are used, please comply with the respective instructions for use. If prescription or non-prescription medication (e.g. ointments, gels etc.) is applied to either the penis or vagina, please check with your doctor or pharmacist for compatibility of the medication with the condom. Caution: Condoms can be damaged by jewellery or body piercings. Each condom should only be used once. Reusing condoms makes them less safe and can lead to unwanted pregnancy and/or infection with HIV or contagious sexually transmitted diseases. Use a new condom in case of any change of sexual activity (vaginal, anal, oral).