SI 0203_032_030

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en BILLY BOY® AQUA LUBRICANT WITH HIBISCUS AROMA
FROMMS BY BILLY BOY® LUBRICANT FRAGRANCE-FREE

150 ml
Medical device – Instructions for use: [i] Read instructions for use carefully before you start using this device. Keep it, you may need to read it again.

Intended purpose: BILLY BOY® lubricant gels are medical devices intended for lubrication of mucosa leading to removing unpleasant feelings due to insufficient mucosal lubrication [i] facilitation of temperature measurement or insertion of tampon into the vagina.
Clinical benefits: Lubrication of the mucosa [i] removal of pain [i] removing of discomfort [i] improving the quality of sex [i] improving the overall quality of life.

Mode of action: The main intended function of the gel in the body or on its surface is not obtained by pharmacological or metabolic means, but by links to its physical properties, especially viscosity and gel consistency. It acts to increase the slipperiness of the surface of the vaginal epithelium and the skin. Character and its mechanism of action is purely physical, and does not act by pharmacological, immunological or metabolic means.

Indications: Insufficient lubrication of mucosa during sexual intercourse [i] pain and discomfort during sexual intercourse [i] fever, menstruation

The device is intended to be used by lay persons – adults, both men and women – or by healthcare professionals during temperature measurement. It can be used with a condom.

How to use: Remove the foil from tube, squeeze out sufficient amount of the gel, based on individual needs, and apply to already deployed condom, directly on the mucosa, or on the tampon or on the thermometer.

Ingredients: Aqua (min. 90%), Glycerin (min. 2%), Hydroxyethylcellulose (min. 1.5%), Sodium Benzoate, Lactic Acid, Aroma (only the perfumed variant contains aroma).

Kontraindikationen: Known hypersensitivity to any ingredient of the product.
Side effects: Allergic reaction.

Warnings: Lubricant gel is water-based. Concomitant use with other vaginal preparations (including vaginal contraception) or rectal preparations consult with your doctor or pharmacist. If irritation occurs, stop using the lubricating gel. If irritation persists, consult your doctor. If the application of the gel does not remove the discomfort from the lack of lubrication, consult your doctor. It does not substitute for contraception. Do not use if the packaging is damaged. Do not use after the expiry date listed on the packaging.

Precautions: Use within 6 months after opening. Any serious incident that has occurred in relation to the use of device should be reported to the manufacturer and the national competent authority and the product should not be used again in this case.

Storage: Keep out of the reach of children. Store in a cool, dry place, between +5°C to +25°C.

Disposal of the device: Dispose of empty packaging in the appropriate container for sorted waste (paper, plastic).

Date of issue of the latest revision of the instructions for use: 07.10.2025

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